

SUPER OB SUSIE® S500.300

Childbirth Training Torso

- Anatomically accurate pelvic landmarks
- Includes full-term neonate featuring palpable landmarks and articulating limbs
- Lifelike cephalic, breech, shoulder dystocia, operative, and C-section deliveries
- Leopold's, version, PPH management, episiotomy repair, and more



Gaumard®
Simulators for Health Care Education

SUPER OB SUSIE® S500.300 | *Childbirth Training Torso*

HANDS-ON OBSTETRIC SKILLS TRAINING

The new OB SUSIE is a realistic childbirth torso designed to aid educators teaching labor and delivery management skills to learners of all levels.

- Pelvic examination
- Urinary catheterization
- Cephalic and breech delivery
- Operative vaginal delivery
- Shoulder dystocia management
- Umbilical cord management
- Placenta delivery
- Rectal medications



REALISTIC ANATOMY

The anatomically accurate pelvis and term neonate allows learners to develop clinical skills transferable to scenario-based exercises.

- Adult-sized lower torso with hip-joint articulation
- Uniform skin with realistic elasticity and soft feel
- Pelvic landmarks include ischial spines and coccyx
- Full-term neonate with articulated limbs (18.5 in, 6 lb)
- Neonate landmarks include fontanelle and sutures

CUSTOMIZABLE AND ADAPTABLE

OB SUSIE's modular design allows you to add new functionalities to meet specific training needs. Choose from a growing list of optional packages.

- Leopold's maneuvers and version (ECV)
- C-section delivery
- PPH management
- Episiotomy repair
- Cervical dilation and effacement trainer
- Delivery Assist Mechanism

ROUTINE DELIVERY COMPETENCIES

- Vaginal examination
- Fetal presentation
- Cardinal movements
- Delivery maneuvers
- Cord management
- Placenta delivery



Palpable pelvic landmarks and dilating cervix



Illustrate descent, rotation, and expulsion




Removable abdomen offers an internal view of the birth

CORD AND PLACENTA MANAGEMENT

- Nuchal cord
- Cord prolapse
- True knots
- Clamp and cut
- Retained placental fragments



Umbilical cord clamping and cutting

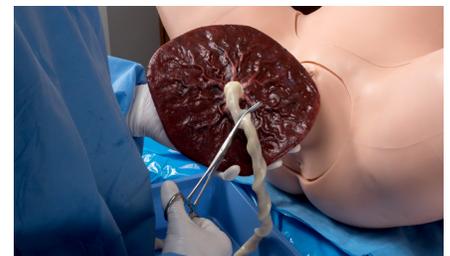

Placenta delivery and examination

SUPER OB SUSIE® S500-300 | *Childbirth Training Torso*

OBSTETRIC EMERGENCIES

OB SUSIE can illustrate realistic shoulder dystocia situations to train advanced delivery maneuvers, teamwork, and communication.

- McRobert's maneuver
- Suprapubic pressure
- Rubin
- Wood's screw
- Arm sweeps



Articulated joints provide realistic resistance



Breech maneuvers: Pinard's, Mauriceau, Ritgen's, Lovset



Forcep indication, application, and traction



Vacuum cup application, suctioning, and traction

BREECH AND OPERATIVE DELIVERY

The neonate's smooth skin and articulated limbs support the use of real instruments and advanced maneuvers.



Soft abdomen and adjustable uterine tone for massage training



Cervix and uterus allow for tamponade placement and hemorrhage control

PPH MANAGEMENT PACKAGE

Train postpartum care and emergency management skills with the optional PPH package.

- Fundal and bimanual massage
- Uterine tamponade placement
- Urinary catheterization
- Suppository administration

OB SUSIE® Childbirth Training Torso

S500.300

The OB SUSIE Childbirth Torso includes everything to get started in one easy-to-use package.



Torso Features

- Adult-sized lower torso: diaphragm to upper legs
- Smooth and supple skin
- Removable pregnant abdominal cover
- Modular cervix and birth canal
- Bony landmarks including ischial spines and coccyx
- Lifelike placenta with detachable umbilical cord
- Patent urethra with bladder access
- Integrated urine bladder (400 mL)
- Integrated blood reservoir
- Articulating hip-joints
- Patent rectum

Neonate Features

- Full term size and weight (18.5 in, 6 lb.)
- Soft full-body skin
- Nasal and oral cavities
- Palpable fontanelle and sutures
- Articulated limb joints
- Detachable umbilical cord

Package Contents

- OB SUSIE Torso
- Pregnant abdomen
- Full term neonate
- Placenta
- (2) Umbilical Cord
- (2) Umbilical Stumps
- (2) Cervices
- (2) Birth Canals
- Mineral Oil
- Talcum Powder
- Blood and urine fill kits

PPH Management Package

S500.300.117

The PPH Management Package enables the training of postpartum care and emergency management skills.



Features

- Realistic post-delivery perineum
- Uterine tamponade placement
- Adjustable uterine tone
- Urinary bladder catheterization
- Suppository administration

Package Contents

- PPH Perineum
- Boggy Uterus
- PPH Cervix
- Blood concentrate

Delivery Assist Mechanism

S500.300.929

The Delivery Assist Mechanism manual lever makes controlling descent and rotation virtually effortless.



Features

- Controlled descent
- Fetal rotation control
- Hold during descent

Package Contents

- Birthing mechanism
- Rear panel
- Fasteners set

Postpartum Episiotomy Suture Trainer

S500.300.133

The episiotomy repair trainers simulate human tissue and support the use of real sutures and instruments.



Features

- Realistic soft skin
- Supports real sutures
- Durable self-healing skin

Package Contents

- Mediolateral left episiotomy perineum
- Midline episiotomy perineum

Coming Soon - Early 2017

- Postpartum 4th degree suture trainer
- Cervical Dilation and effacement trainer
- Leopold's and version (ECV) trainer
- C-section delivery package

Distributor:



GM MEDICAL
Product Specialist Brian Vester
+4531552280
BV@gm-medical.com



Gaumard®
Simulators for Health Care Education